



## INSTRUCTIONS FOR CLEANING EYEGLASSES WITH ANTI REFLECTIVE COATING

Lenses with AR Coating : For best results for cleaning the lenses please do the following:

1. Wash the lenses with cold/lukewarm water to remove the dust particles (with the dust removed there is nothing to scratch the lenses).
2. Drop a small drop of dish washing liquid on your wet fingers and gently rub the wet lenses.
3. Rinse with cold/lukewarm water.
4. Dry with a clean 100% cotton or microfiber cloth. Lenses with hydrophobic coating have just 1 or 2 drops of water to blot.
5. Do not use any other eye cleaners or other cleaning products with any chemicals, they will destroy the AR coating that is on the glasses.

**Thank you for your order!** We take pride in offering a wide variety of leading-edge eye wear at reasonable prices. **We hope that you enjoy your new driving glasses.**

**We hope that you enjoy your new prescription glasses.**

**We have included a FREE case and microfiber pouch to assist in protecting your glasses.**

**We want to know what you think of our glasses. Please send us your product comments/testimonials using our Contact Us page at: <http://www.vseyewear.com/store-pres/>**

**We also are offering the following special offer: 10% off on your next purchase with our company. Use coupon code RP10. You can use this coupon to purchase any of our items.**

